



Saliva Sidekick™ Saliva Ejector Tube Holder

1. When sterilized and ready to use, position your saliva ejector tube under the tube holder and press it up into the holder until it is secure.
2. Slide the Saliva Sidekick into the magnetic track on the Bite Buddy, ensuring that it is securely in place.
3. Position the Bite Buddy with the Saliva Sidekick into the mouth according to the Bite Buddy instructions.
4. The Saliva Sidekick can be removed with the Bite Buddy or slid from the track while the Bite Buddy remains in the patient's mouth. Without pulling upwards or downwards, gently slide the saliva ejector tube with the Saliva Sidekick straight out of the Bite Buddy track.

Sterilization Instructions:

Remove from packaging and sterilize all products according to these instructions before use. Sterilization should be performed using chemical methods or via autoclave. Insufficient water to the autoclave could cause parts to melt. Cold Sterilization is the recommended method of sterilization.

Cold Sterilization:

1. As soon as possible after use, soak and rinse your Ascentcare instruments in warm water to remove any blood or tissues.
2. Select a liquid chemical agent registered with the EPA as a "sterilant."
3. Follow the sterilization instructions of your sterilant liquid. Most chemical/cold sterilization solutions render instruments sterile only after 10-hour immersion.
4. Rinse, dry and store in a sterile container if not used immediately.

The Saliva Sidekick is latex free.

As with all products containing magnets, please take care to avoid inadvertently deactivating such devices as pacemakers, ICD's and other implanted medical devices.

Be sure to take all necessary precautions to avoid dislodgement or aspiration by following the instructions in this pamphlet.

Ascentcare's products are protected by US Patents.
For patent information, visit:

www.ascentcaredental.com/patents

Ascentcare™
Dental Products

(800) 500-0533

www.ascentcaredental.com
service@ascentcaredental.com