



## **Bite Buddy™**

### Mouth Prop

#### ***Instructions:***

1. When sterilized and ready to use, floss may be tied around the Bite Buddy for quick retrieval if needed.
2. Place in mouth over back molars using Placement Pal or index finger.
3. Do not place over front teeth.
4. To remove, use Placement Pal or index finger.



[www.ascentcaredental.com](http://www.ascentcaredental.com)  
(800) 500-0533

## **Sterilization Instructions:**

Remove from packaging and sterilize all products according to these instructions before use.

Sterilization should be performed using chemical methods or via autoclave. Chemical sterilization is the recommended method of sterilization for added longevity of your products.

#### ***Autoclave Sterilization:***

Please note: Insufficient water to the autoclave could cause parts to melt.

1. Products should be cleaned as soon as possible after use to prevent soils from hardening. Remove all visible debris prior to sterilizing.
2. Process product through steam autoclave following the autoclave manufacturer's instructions. Do not exceed a temperature of 136°C/275°F.
3. Store only in FDA approved sterilization pouch.

#### ***Chemical Methods:***

1. As soon as possible after use, soak and rinse your Ascentcare instruments in warm water to remove any blood or tissues.
2. Select a liquid chemical agent registered with the EPA as a "sterilant."
3. Follow the sterilization instructions of your sterilant liquid. Most chemical sterilization solutions render instruments sterile only after 10-hour immersion.
4. Rinse, dry and store in a FDA approved sterile container if not used immediately.

The Radical Reflector is latex free.

As with all products containing magnets, please take care to avoid inadvertently deactivating such devices as pacemakers, ICD's and other implanted medical devices.

Ascentcare's products are protected by US Patents.  
For patent information, visit:  
[www.ascentcaredental.com/patents](http://www.ascentcaredental.com/patents)

**(800) 500-0533**  
[service@ascentcaredental.com](mailto:service@ascentcaredental.com)